

Chemical Recovery Cartridge, Model II

Chemical Recovery Cartridge, Junior Model II

Installation Instructions

- **Use qualified personnel to install this equipment**
- You **must** fill the cartridge with water at the installation. This minimizes channeling and clogging.
- **The solution flows by gravity in the system. The drain tubing must always slope down from the processor fixer overflow outlet, through the cartridge, to the drain**

Note: *Some processors may require an adaptor kit; refer to your processor manual for the details*

Contents – 1-bucket, 2-90° elbows and 2-plugs. Not included $\frac{3}{4}$ inner diameter (I.D.) vinyl tubing.

No patent license is granted by the information.

Carestream Health reserves the right to change this information without notice, and makes no warranty, express or implied, with respect to this information. Carestream Health shall not be liable for any loss or damage, including consequential or special damages, resulting from any use of this information, even if loss or damage is caused by Carestream Health's negligence or other fault.

Warning: *Gloves and protective eyewear must be worn when working with chemistry.*

Single Cartridge Installation

1. Remove the cartridge and fittings from the box. Save the box and the plug fittings for use when returning the cartridge for refining.
2. Included, is a package containing $\frac{1}{2}$ " x $\frac{3}{4}$ " elbow hose adaptors. The threaded ends have been wrapped with pipe-sealing tape.

Important: *To prevent leaking, do not remove the pipe-sealing tape. For all fittings – **hand tighten only.***

3. Thread an elbow hose adaptor into the "IN" port, on the upper-side of the cartridge.
4. Connect the fixer overflow outlet from the processor to the elbow hose adaptor that is threaded into the cartridge "IN" port, using $\frac{3}{4}$ I.D. and tubing.
5. Thread the remaining elbow hose adaptor into the "OUT" port that is in the upper-side of the cartridge.
6. Connect one end of the length of $\frac{3}{4}$ I.D. tubing to the elbow hose adaptor that is threaded into the cartridge "OUT" port, and direct the other end to the floor drain.

Multiple (Series) Cartridge Installation

1. For cartridge #1, follow steps 1-5 in the "Single Cartridge Installation" section. Place cartridge on a platform that raises it approximately 10 cm (4 inches) above the floor.
2. For cartridge #2, follow steps 1-3 and 5 in the "Single Cartridge installation" section. Place #2 on the floor close to #1. Turn the ends of the elbow hose adapters in the #1 "OUT" port and #2 "IN" port, so that they point toward each other.

Important: *Tubing should be just long enough to reach, and not sag or bend. Support tubing as necessary.*

3. Connect the "OUT" port of the #1 to the "IN" port of #2, using a short length of $\frac{3}{4}$ " ID tubing.
4. Connect one end of a length of $\frac{3}{4}$ " ID tubing to the elbow hose adaptor that is threaded into the "OUT" port of cartridge #2, and direct the other end to the floor drain.

Electrolytic Unit and Tailing Cartridge Installation

1. Follow steps 1-3 and 5 in the "Single Cartridge Installation" section.
2. Connect the outlet or overflow from the electrolytic unit to the elbow hose adaptor that is threaded into the cartridge "IN" port, using $\frac{3}{4}$ " ID tubing.

Important: *Make sure tubing is not kinked.*

3. Connect one end of the length of $\frac{3}{4}$ " ID tubing to the elbow hose adaptor that is threaded into the cartridge "OUT" port, and direct the other end to the floor drain.

For more information, call **800.933.8031** or visit us on the web at **www.carestreamdental.com**.